

Medical Infrared Thermometer



One Second Measurement



Start with one press



32 Sets of Memory



Accuracy Measurement



Audible and Visual Alarming



Non-contact Design



LCD Display



Simple Operation



BESTMAN

BFT-866

Feature

- 1、 Display of HD Larger screen, three color backlight(green、 white and red),display measured temperature value、 temperature unite、 battery and etc.
- 2、 With super memory store function, can display 32 sets of data of examination of temperature, intelligent and can be tracked.
- 3、 With function of measuring temperature of human begins and objects, Convenient for daily measurement.
- 4、 The device can switch between Chinese and English; between Celsius and Fahrenheit to meet individual needs.
- 5、 Intelligent acousto-optic indicator and automatic shutdown function with simply operation.
- 6、 The measuring and displaying temperature are accurate and reliable.

Structure

Screen Size	24mm×30.5mm
Display Resolution	0.1°C/°F
Accuracy	±0.2°C/±0.4°F
Operating Voltage	DC 3V 2×AA
Auto Power-off	10s±1s
Storage and Transpotation Environment/Relative Humaidity	0-93%RH non-condensing
Temperature Range	32.0°C~42.5°C (89.6°F~108.5°F)
Maximum Permissible Errors	Within the scope of: ±0.2°C (±0.4°F) Out of the scope of: ±0.3°C (±0.6°F)
Service life	3 years (Except for the consumable parts)

One-key start design, and one second quick testing with stable consistency

Non-contact infrared measurement, 3-5 cm distance, one second quick test, avoid cross infection



Green Backlight
< 37.4 normal



Yellow Backlight
37.5-38 little high



Red Backlight
> 38.6 high

Three Colors Backlight Indication

Multi- purpose Infrared Thermometer

A button to switch body temperature and object temperature. Can test the temperature at anytime and avoid causing harm to baby because of high or low temperature.



Normal Status: Object and body temperature conversion
History Status: Turn to last testing data
Setting Status: Adjust the setting data




Short Press: Enter to check the history data
Long Press For 5 Seconds: Enter to the setting page
Setting Mode One(F1): Alarming temperature setting page.
When the measured temperature is higher than the alarming temperature, the backlight will turn into red color.
Setting Mode Two(F2): °C/°F conversion
Setting Mode Three(F3): Chinese/English Conversion
Setting Status: Press it to choose the setting mode.
When the setting has been done, press again to turn off the device.



Normal Status: Sound switches
History Status: Turn to the next testing data
Setting Status: Adjust the setting data

Product details

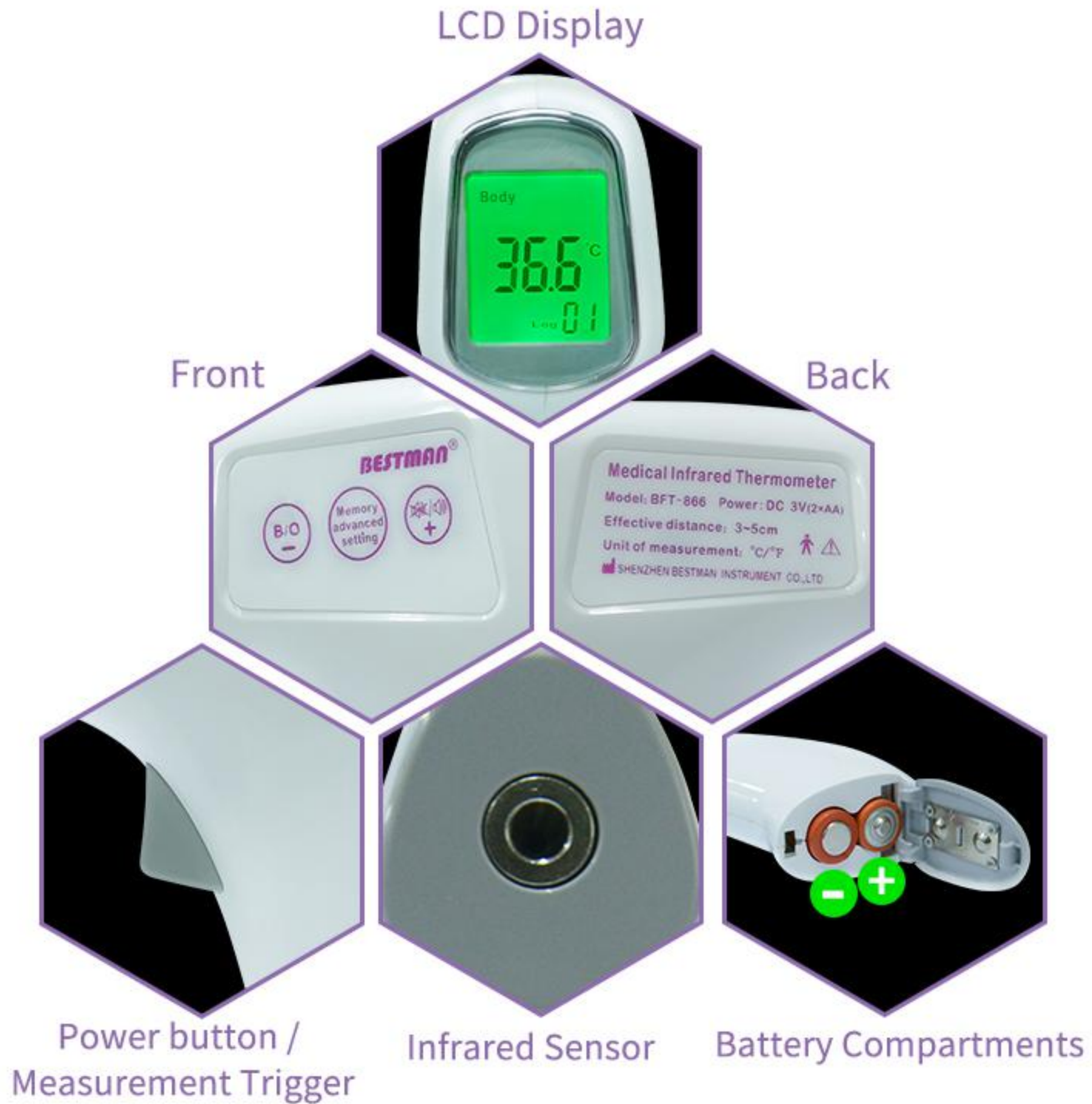


Energy Saving

When there is no operation after the test is completed, the thermometer will automatically shut down within 10 seconds.



Structure



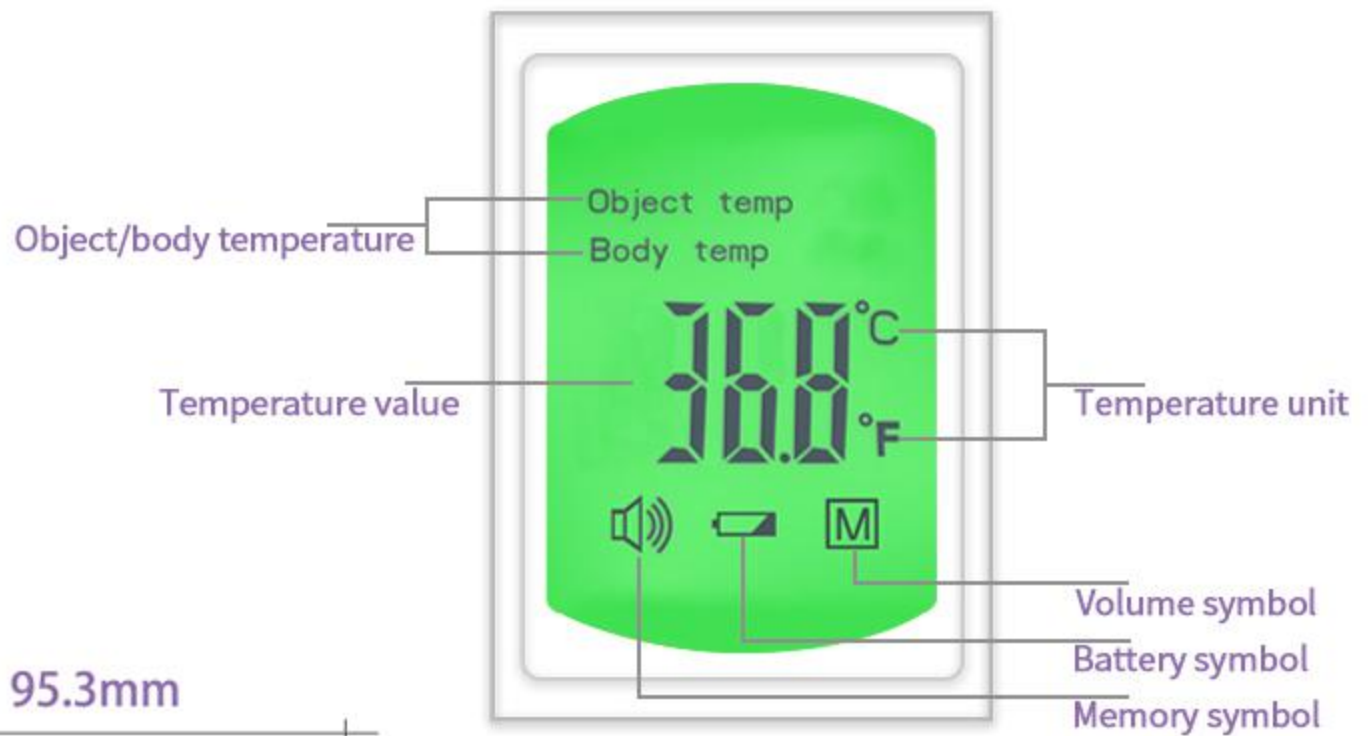

Dimension



SHENZHEN BESTMAN INSTRUMENT CO.,LTD

www.szbestman.cn

Add: 8/F,YiFang Bldg,No.315 Shuangming Ave,Dongzhou Community, Guangming St.,Guangming Dist., Shenzhen, Guangdong,518107, China

Tel: +86-755-26411423

Fax: +86-755-26495167